

# QUICK REFERENCE GUIDE


# GENDRON Sentinel II Bariatric Low Bed - Model 3875

This quick reference guide is not intended to be a substitute for the user manual. Read the entire user manual before using this device.

## Patient Pendant

Press and hold button to operate each function

- 1) Head Up ↑ and Down ↓
- 2) Knee Up ↑ and Down ↓
- 3) Nurse Call

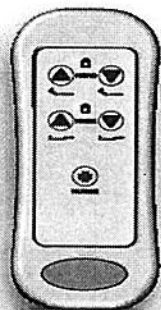

## Electric Central Brake/Steer Function

Brake/Steer controls are located at both head end siderails and at the footboard caregiver controls. An LED will be lit above the button to indicate status.

Neutral Position – casters unlocked


Steer Position – foot end casters are locked inline with bed

Brake Position – all casters locked



## Caregiver Controls

Caregiver controls are located at the foot end of the bed, on the side opposite of the rotating scale display.

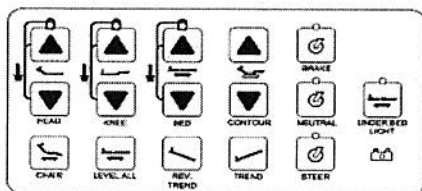

## Lockout control

To lock/unlock a function from being used press and hold the up and down arrow button of the function to be locked/unlocked. A yellow LED is lit above the buttons of locked functions.

## Caster

Press either the BRAKE, NEUTRAL or STEER button to activate the desired caster function.

## Head/Knee/Bed/Tilt (trend/reverse trend)

Press and hold the button of the function desired + the UP or DOWN button.

## Cardio chair position

Press and hold the CHAIR button.

## Level All

Press and hold the LEVEL ALL button.


## Under bed light

Press the UNDER BED LIGHT button to turn the light on or off. A yellow LED is lit above the button when the light is on.

## Siderail Operation

Siderails are located on each side of the bed. Siderails have a 3-position swing down operation.

To raise siderail, grasp top section of rail and lift. Allow release pin to snap into the locked position.

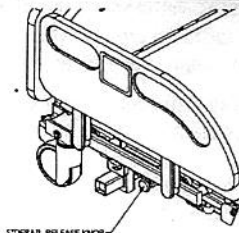


**Caution:** Gently pull on the siderail to verify that it is latched in position.

To lower siderail, push release knob and lower siderail.

## CPR Release

For emergency lowering of head section pull and hold the CPR release handle. The CPR release handle is located at the head end of the bed on both the left and right side.

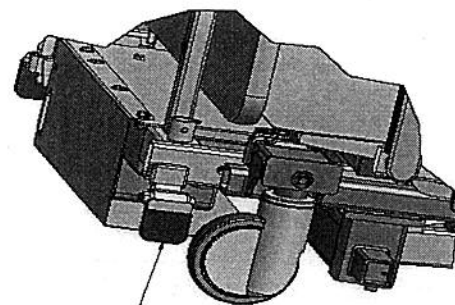


**Caution:** Always allow head section to fully lower when using CPR release.

## Note:

1. CPR release may not operate unless a patient is lying on the bed.
2. After activating CPR function it may be necessary to drive the actuator inward (head down) to reset the actuator before raising the head section.

## Side Extension

Using the deck extensions the bed width can be changed from 38" to 48". To adjust the side extensions pull the black knob that is below each extension and move to the desired position.

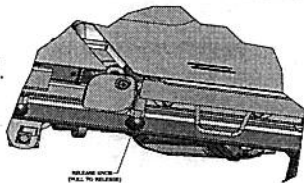


**Caution:** Gently pull on the extension to verify that it is locked in position.

**Caution:** Ensure that all side extensions are adjusted to the same width. Equipment damage may occur during deck articulation.

**Caution:** Ensure that deck extensions are adjusted to properly fit the mattress. Improperly adjusted extensions may create dangerous gaps between the mattress and siderails.

## Foot Extension

Using the foot extension, the bed length can be adjusted from 80" to 84" to 88". To adjust the foot extension, pull the black knob that is at the center of the extension (below the footboard) and move to the desired position.

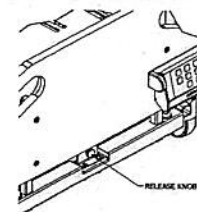

**Caution:** Gently push/pull on the extension to verify that it is locked in position.

**Caution:** Ensure that the extension is adjusted properly to fit the mattress. An improperly adjusted extension may create a dangerous gap between the mattress and footboard.

# QUICK REFERENCE GUIDE

# GENDRON Sentinel II Bariatric Low Bed - Model 3875

This quick reference guide is not intended to be a substitute for the user manual. Read the entire user manual before using this device.

## Scale

The scale can be used to monitor the patient's weight, change in weight and to alert the caregiver when the patient exits the bed.

Menu is used to edit system setup.

Zero Weight press and hold to change displayed weight to zero. Do not press when patient is in the bed.

Zero Weight Change press and hold to change the displayed weight change to zero.

Weigh wakes the display. Also displays current weight/weight change data.

Freeze lets items be added or removed from the bed and not have an effect on the shown patient weight. Also, if a know error exists, lets the caregiver manually correct the shown patient weight. Press and hold to freeze.

Up/Down arrows. Use up arrow to change menu selection. Use down arrow to toggle value. When in Freeze mode, arrows can be used to make manual adjustments to the weight value.

Bed Exit Alarm press to enable or disable alarm. Use the menu button to program the alarm parameters. The facility nurse call can be activated by the bed exit alarm if desired by using either of the connectors at the head end of the bed.

The parameters listed below can be changed through the Menu.

Bed Exit Alarm on 0-45% of patient weight in 5% increments. 5-15% high sensitivity, 20-30% medium sensitivity, 35-45% low sensitivity.

Bed Exit Alarm on time 30-300 seconds in 30 second increments. Alarm will sound for the amount of time desired between 30-300 seconds.

Units Kg or LB

Backlight intensity 0-10 in 1 unit increments

### To edit system setup:


- Press Menu
- Use Up arrow to change menu selection
- Use Down arrow to toggle value
- Press Freeze to save changes or
- Press Weigh to exit without saving changes

### To adjust bed exit alarm sensitivity and enable:

- Zero Weight Change
  - Press Menu
  - Use Up arrow to change menu selection to Bed Exit
  - Use Down arrow to toggle value
  - Press Freeze to save changes or
  - Press Weigh to exit without saving changes
  - Press Bed Exit to enable alarm
- Note: Patient must be in the bed before enabling the alarm.


### To correct shown patient weight

If the shown patient weight is known to be wrong press and hold the Freeze button, then use the Up and Down arrows to correct the shown weight. When adjustments are complete press the Freeze button to save changes and exit, or press the Weigh button to exit without saving changes.

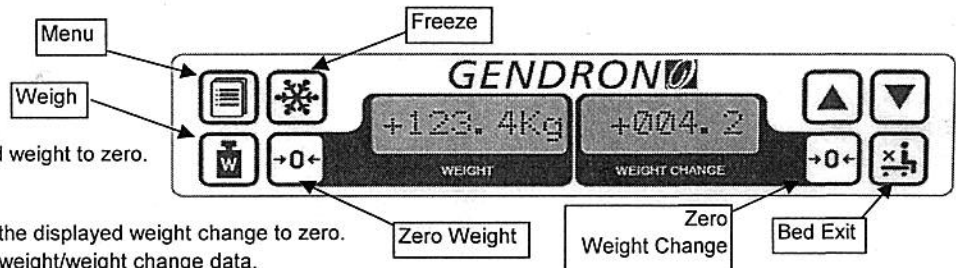

## Brake Pedal Override on Caster Lock

A manual brake pedal is provided to allow an override of the caster lock actuator upon any failure of that device. Remove the brake pedal provided in the storage area below the head end right side of the bed on the lower frame and attach it to the center shaft of the caster lock mechanism. Remove the pin in the actuator and the brake pedal can now manually set the casters in their lock, steer and swivel positions

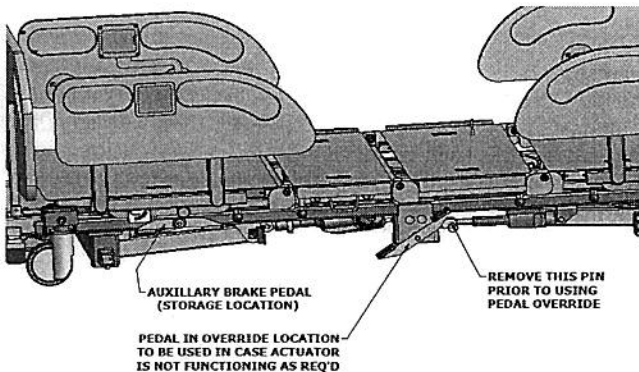

## Battery Backup

Battery backup allows full use of bed functions when AC power is not available for a limited time.

⚠ Caution: Disconnect battery to prevent discharge during storage. Failure to do so will reduce battery life.

Use a small blade screwdriver to release the retainer through the access hole in the controller cover.

