


BED OPERATION

Bed Positioning – Bed functions are operated with either an 8 or 10 button pendant. A 10 button pendant with lock out feature is shown below. Button features on the pendant are clearly identified with universal icons.

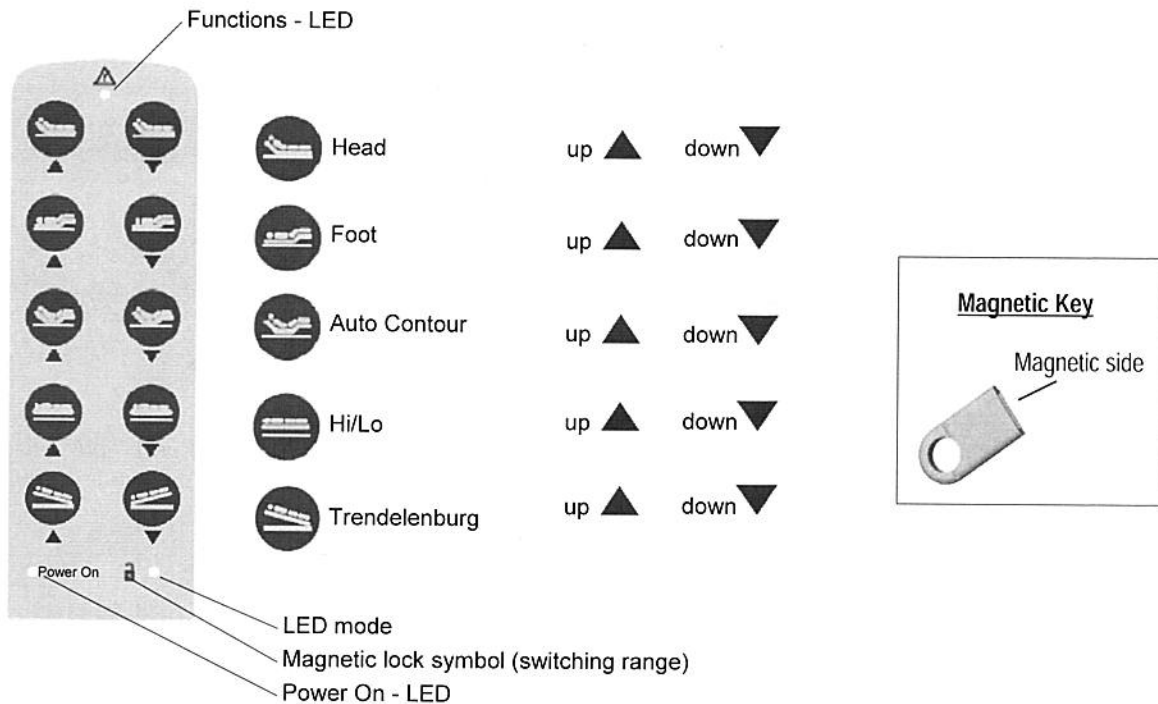

Operation of the Lock Out Pendant

By passing the magnetic side of the magnetic key over the lock symbol, three modes can be released or locked.

- To put the Lock Out Pendant into operation, connect it to the MCL Control Box through the "Y" cable. The keyboard of the pendant is completely blocked (mode 1). The LED next to the lock symbol **does not glow**.
- Passing the magnetic side of the magnetic key once over the lock symbol releases the resident comfort functions (mode 2). The pendant will stay in this mode until the magnetic key is once again passed over the lock symbol. The LED next to the lock symbol **glows green**.
- Passing the magnetic side of the magnetic key once again over the lock symbol releases the HI/LO and Trendelenburg/Reverse Trendelenburg functions (mode 3). The LED next to the lock symbol **glows amber**.

While in mode 3 if no button is pressed after approximately 50 seconds of the pendant having last been actuated, mode 1 is reset.



Caution

Keep the Lock Out Pendant away from magnetic fields which could change the Pendant Mode.