


Zenith Quick Reference Guide

- Caster brakes should be kept locked when in use.
- Consider the use of floor pads to increase patient safety.

HAND CONTROL FEATURES

PATIENT CONTROLS: Each bed is equipped with a pendant type hand control as standard equipment. Each control function (head, knee, and bed) has two clearly identifiable switches for up and down travel. The controls are designed to prevent damage to the motor in the event that both the up and down functions are activated simultaneously.

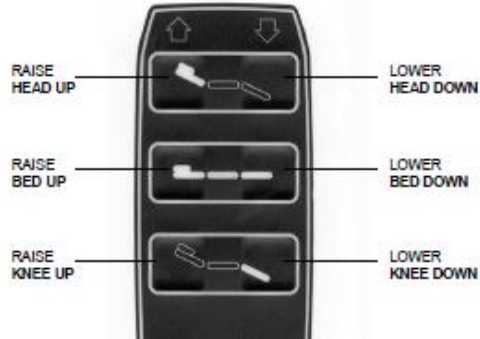

Zenith Specifications			
Overall Length	87"	Inside Length Head to Footboard	82"
Overall Width (with rails)	40.5"	Deck Height (High)	30"
Overall Width (with boards)	35.88"	Deck Height (Low)	7.95"
Deck Width	35"	Maximum Head Elevation	70 deg
Deck Surface Length	79.75"	Maximum Knee Elevation	30 deg
Weight Capacity	500 lb	Deck to Top of SideRail	12"